

User Seal Check Instructions

1



Hold respirator in hand with molded nose contour (narrow end) at finger tips, allowing headstraps to fall below hand.

Discard respirator when damaged, soiled or causing noticeably increased breathing resistance or discomfort. Respirators cannot be disinfected.

2



Place respirator under chin with molded nose contour (narrow end) up. Nose cushion must not be folded inside respirator. Raise top strap to top back of head. Pull shorter bottom strap over head, below ears, to rest around the neck. (Do not wear with only one strap because it may affect fit.) Adjust respirator for comfortable fit.

3



Fit-check the seal each time the user enters a contaminated area. Cover front of respirator by cupping both hands. **INHALE SHARPLY.** A negative pressure should be felt inside the respirator. If any leakage is detected at respirator edges, reposition respirator and/or select another size and perform fit test. Repeat until sealed properly, otherwise see your supervisor. (Entry into a contaminated area with an improper fit may result in sickness or death.)