

Donning Instructions



Step 1 Open Mask

Grasp nose piece and chin piece. Snap open to activate the Magic Arch®. A "pocket" is formed in the mask. Note: mask must be fully open.



Step 2 Conform To Face

Place chin in mask "pocket." Place bottom headband below ears and top headband at crown of head. Conform nose piece to nose and facial contours.



Step 3 Fit To Face

Holding mask at nose piece, gently pull down under chin so mask fits snug against face.



Step 4 Positive Facial Lock

Pinch any excess material together under the chin and twist the Twist Seal® chin piece.

Step 5 Fit Check Place both hands along the edges of the mask and exhale. If air escapes at the nose, adjust the nose piece. If air leaks at the edges, untwist the Twist Seal® chin piece and repeat Step 3.